



## **The Door Switch™ Keypad Instructions**

When an “EVENT” occurs, The Door Switch™ audible alarm(s) will sound and the designated strobe light in the corridor will flash. The keypad will show the Door # and Main or Bathroom Door.

**Step One:** Check the corridors for a flashing strobe light to determine the correct rooms’ location and help the patient.

**Step Two:** Turn the keyswitch by the room in alarm, to silence the audible alarm. (Strobe light will continue to flash).

**Step Three:** At the keypad, press [1] + [2] + [3] + [4] + [#] + [7] + [1] to deactivate the Strobe Light.

**Step Four:** At the keypad, press [1] + [2] + [3] + [4] + [1] (OFF) to clear the keypads read out and return it to the default screen reading “THE DOOR SWITCH™ - SYSTEM ENABLED”.

Note: If “Trouble” message appears on the keypads read out, press [1] + [2] + [3] + [4] + [1] (OFF) two times to return to the default screen reading. If “Trouble” message continues, check the system for tampering and/or required maintenance.